



# How to assemble and adjust

the XXL-Rehab Patient/Dinner chair

## Assembling

### Assembly of the XXL-Rehab Patient/Dinner chair

The XXL-Rehab Patient/dinner chair consists of two parts, the chair frame and the backrest.

1. Put a washer on each bolt. Fastened the backrest to the chair frame by inserting the four bolts and washers into holes, two on each side. Tighten them well with the accompanying hexagon allen key.

### Adjustment of the seat depth and backrest

1. The seat depth can be adjusted into three positions. Choose two holes (front, middle or back) on the backrest for the desired seat depth. See the instruction above.

### Adjustment of the height settings of the legs

1. The height of the legs can be adjusted into six positions by loosening and tightening the four bolts already mounted on the legs. Use both hexagon allen keys when adjusting.

### Tools to use

Two 5 mm hexagon allen keys, four hex sockets allen key bolts, four washers.



## Warning

The XXL-Rehab Patient/Dinner chair may only be assembled and adjusted by qualified staff who has been instructed in the proper use of the XXL-Rehab Patient/Dinner chair. Incorrect assembling and adjustment can injure the user.